Charger Port

Backlight

Display

Power Button
ON/OFF

P/N 571394-0001 Rev. AB 2014-09-18
CS-1405 PL-US © Orthofix Holdings, Inc.

Cervical-Stim® Quick Instructions:

1. Press the **Power Button** to start.
2. The **LCD Display** will show start-up codes, then the 4-hour daily treatment begins.
3. Wear the stimulator for **4 hours** every day.*
4. After treatment is complete, the **LCD Display** will show 3 dashes - - - and power off.
5. To recharge, plug the charger into **Charger Port** and AC outlet.

*Clinical trial data demonstrates that fusion success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P030034; FDA Approved December 2004.*

See the instruction manual for full usage information.

Patient Services
800-535-4492
www.bonestimulation.com

Charger Port

Backlight

Display

Power Button
ON/OFF

P/N 571394-0001 Rev. AB 2014-10-01
CS-1405 PL-US © Orthofix Holdings, Inc.

Cervical-Stim® Quick Instructions:

1. Press the **Power Button** to start.
2. The **LCD Display** will show start-up codes, then the 4-hour daily treatment begins.
3. Wear the stimulator for **4 hours** every day.*
4. After treatment is complete, the **LCD Display** will show 3 dashes - - - and power off.
5. To recharge, plug the charger into **Charger Port** and AC outlet.

*Clinical trial data demonstrates that fusion success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P030034; FDA Approved December 2004.*

See the instruction manual for full usage information.

Patient Services
800-535-4492
www.bonestimulation.com

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