Physio-Stim® Quick Instructions:

1. Press the **Power Button** to start.
2. The **LCD Display** will show start-up codes, then the 3 hour daily treatment time.
3. Wear the stimulator over treatment site for **3 hours** every day.*
4. After treatment is complete, the **LCD Display** will show 3 dashes --- and power off.
5. To recharge, plug the charger into the **Charger Port** and AC Outlet.

*Clinical trial data demonstrates that fusion success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P850007; FDA Approved February 1986.*

See the instruction manual for full usage information.

PN-20116508 Rev.AA 2014-09-19
PS-1423 PL-US © Orthofix Holdings, Inc.

Patient Services
800-535-4492
www.bonestimulation.com

Physio-Stim® Quick Instructions:

1. Press the **Power Button** to start.
2. The **LCD Display** will show start-up codes, then the 3 hour daily treatment time.
3. Wear the stimulator over treatment site for **3 hours** every day.*
4. After treatment is complete, the **LCD Display** will show 3 dashes --- and power off.
5. To recharge, plug the charger into the **Charger Port** and AC Outlet.

*Clinical trial data demonstrates that fusion success rates were significantly higher for patients who wore the device every day for the full treatment time. *Source: Premarket Approval P850007; FDA Approved February 1986.*

See the instruction manual for full usage information.

PN-20116508 Rev.AA 2014-09-19
PS-1423 PL-US © Orthofix Holdings, Inc.

Patient Services
800-535-4492
www.bonestimulation.com

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